

# The **Accu-Cut**<sup>®</sup> Osteotomy Guide System

Patent # 5,843,085

## Precision and Consistency

The BioPro<sup>®</sup> Accu-Cut Osteotomy Guide System provides precise and repeatable osteotomies for all types of bunion surgery.



## Features & Benefits

- ④ Simple, single-use sterile packs
- ④ Custom saw blades for high precision cuts
- ④ Easily reproducible, osteotomy after osteotomy
- ④ The variety of available guides allow for single, dual and tri-plane correction

MORE INFORMATION AVAILABLE AT:  
[www.bioproimplants.com](http://www.bioproimplants.com)  
[info@bioproimplants.com](mailto:info@bioproimplants.com)

BIOLOGICALLY ORIENTED PROSTHESES  
**BIOPRO**

Manufactured  
In the U.S.A.

# The BioPro® Accu-Cut® Osteotomy Guide System



## Chevron (Austin) Guide

The standard chevron guide offers a 55° angle for precise cuts on a classic, predictable osteotomy.



## DMAA Guide

The DMAA Guide allows for correction of DMAA with one simple guide. The system includes a chevron guide with a second dorsal slot, angled at 10°. This allows for lateral transposition plus rotation of the head.



## Youngswick Guide

The Youngswick guide offers double dorsal slots, spaced 1mm, 2mm, or 3mm apart. This allows the surgeon to shorten and plantarflex the metatarsal head by 1, 2, or 3 mm.



## Scarf (Z) Guide

The Scarf guide is a "Z" shaped guide to assist in the popular Scarf procedure. The system includes two guides, a large and a small, with each guide having multiple slots for the dorsal and plantar cuts. This allows for customization based on the anatomy of the patient's metatarsal.



## Long Arm Guide

The LongArm guide is a 45° degree chevron with a long dorsal arm. This offers more surface area along the dorsal cut for multiple screw fixation. Additionally, the guide offers three proximal mounting holes, spaced 11° apart, allowing adjustment of the cut angle.



## 2-Stage Guide

The 2-Stage guide (Reverdin-Gerbert) allows for correction of PASA, or DMAA. The system includes a standard chevron guide (stage 1) and angled guides (stage 2) in 5°, 10°, and 15°. This allows for correction up to 15°.

Visit [www.bioproimplants.com](http://www.bioproimplants.com) to download surgical techniques.



Description	Item #	Replaces Item #'s		
Austin/Chevron Guide	19505	16802	16911	17721
Youngswick 1mm Guide	19506	16803	16912	17733
Youngswick 2mm Guide	19507	16804	16913	17734
Youngswick 3mm Guide	19508	17300	17306	18354
2-Stage PASA/DMAA Guide	19509	16957	16958	17735
LongArm Chevron Guide	19510	19262	19261	19264
Z (Scarf) Guide	19511	17792	17793	17795
DMAA Guide	19720	n/a	n/a	n/a



Scan to visit website

BIOLOGICALLY ORIENTED PROSTHESES

**BIOPRO**

800.252.7707 [www.bioproimplants.com](http://www.bioproimplants.com)  
2929 Lapeer Road - Port Huron, MI 48060



Brochure No. 16932 rev 07